Millipore.

SigmaAldrich.com

User Guide

Milllex®-GP/HP Filter Unit (Sterile)

with Millipore Express® PES Membrane

REF

SLGPM33RS

SLHPM33RS

English

Français

Italiano

Deutsch

Español

Português

Nederlands

Dansk

Svenska

Latviski

Lietuviškai

Magyar

Česky

Polski

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Norsk

The life science business of Merck KGaA, Darmstadt, Germany operates as MilliporeSigma in the U.S. and Canada.

PRO5336 Rev 06/23 Date of issue: 2023-06

Symbol	Definition	Symbol	Definition
	Sterilized using irradiation		Use-by date
	Non-pyrogenic		Date of manufacture
	Consult instructions for use		Manufacturer
	Do not use if package is damaged and consult instructions for use		Caution
	Do not resterilize		CE conformity marking
	Do not re-use		Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare professional.
	Catalogue number		Swiss authorised representative
	Batch code		Unique device identifier
	Temperature limit		

Made in Ireland

Merck Millipore Ltd.
Tullagreen, Carrigrohilly,
Co. Cork, Ireland
T45 KD29
an affiliate of Merck KGaA,
Darmstadt, Germany

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

The initial M, Millipore, Millex, Millipore Express and Sigma Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.

© 2002-2024 Merck KGaA, Darmstadt, Germany and/or its affiliates. All rights reserved.

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• Non-pyrogenic

Intended Purpose

The Millex®-GP/HP Filter Unit (Sterile) are medical devices intended for use as syringe filters to sterilize and/or clarify low volume solutions in direct patient care and pharmacy admixture applications. Devices are intended for medical and/or laboratory professional use only.

Introduction

This document provides compatibility information, operating steps, and specifications for the Millipore Express® polyethersulfone (PES) family of sterile filters. These filters remove microorganisms, particles, precipitates, and undissolved powders larger than the membrane's rated pore size from solutions for clinical use. In the hospital pharmacy, these filters with Millipore Express® PES membrane can be used for the sterile filtration (GP) and/or clarification (GP/HP) of small volumes of protein pharmaceuticals, diagnostic imaging agents, chemotherapeutics, aqueous solutions, or water during admixture preparation. Direct patient care applications include sterilization (GP) and/or particulate removal (GP/HP) from epidural and other liquid anaesthetics as well as from irrigation solutions used in ophthalmic, etc, and other surgical procedures.

Chemical Compatibility

These filters are compatible with most aqueous solutions. Compatibility of solution should be tested before use. Visit [SigmaAldrich.com/MedMillex](#) for more information.

WARNINGS

- Filter solutions with a temperature limit of 45 °C (113 °F).
- Syringe filters are for manual use only; do not use on automated systems.
- Not suitable for filtering high viscosity solutions or blood.
- Do not filter solutions containing 5 mg or less of active drug materials unless binding studies have been performed.
- To ensure sterility, do not use this product if the package is damaged.
- Use aseptic technique to maintain sterility removing product from primary package.
- Do not use with syringes smaller than 10 mL because pressures in excess of the maximum pressure rating may be reached, potentially causing damage to the filter and/or personal injury.
- Do not use for direct neuraxial applications which may come in contact with cerebrospinal fluid.
- Sudden loss of pressure could indicate failure of the filter.
- Single use only; do not re-use or resterilize.
- This membrane is not bi-directional. Do not use the filter to filter in reverse direction.
- Make sure to wet the filter membrane thoroughly before injecting the solution; improperly wetted filters can become airlocked.
- Not suitable for use with medical pressure infusion equipment or gravity fed intravenous (IV) administration sets.
- Discard appropriately after single use. See "Disposal" below.

Directions for Use

1

Wearing gloves using aseptic technique: fill syringe with solution to be filtered.

2

Remove cover from package.

3

Attach suitable syringe to filter inlet and remove assembly from package.

4

Hold syringe with filter pointing up and top off by pushing a few drops through to wet filter. Excess solution may be hazardous and should be disposed with care.

5

Depress syringe plunger to filter solution into suitable container or compatible attachment on outlet.

Specifications and Performance Characteristics

Membrane	Hydrophilic Millipore Express® polyethersulfone (PES)
Pore size	Millex®-GP filter: 0.22 µm Millex®-HP filter: 0.45 µm
Housing	Modified acrylic copolymer (MMA)
Dimensions	
Inlet to outlet	27 mm (1.06 in.)
Diameter	33 mm (1.30 in.)
Filtration area	4.52 cm² (0.70 in²)
Temperature limit	45 °C (113 °F)
Pressure limit at 21 °C	10.3 bar (150 psi) inlet maximum
Filtration volume	10 mL to 200 mL
Hold-up volume	≤ 0.1 mL after air purge
Sterilization method	Gamma irradiation
Connections	Female Luer-Lok™ inlet; male Luer-slip outlet
Flow rate at 2.1 bar (30 psi), 21 °C	Millex®-GP filter: ≥ 150 mL/min Millex®-HP filter: ≥ 300 mL/min

Notice

We provide information and advice to our customers on application technologies and regulatory matters to the best of our knowledge and ability, but without obligation or liability. Existing laws and regulations are to be observed in all cases by our customers. This also applies in respect to any rights of third parties. Our information and advice do not relieve our customers of their own responsibility for checking the suitability of our products for the envisaged purpose.

Contact Information

For the location of the office nearest you, go to [SigmaAldrich.com/offices](#).

Disposal

Follow precautions for disposal of items contaminated with potentially infectious or hazardous biological material according to all applicable international, federal, state, and local regulations.

Technical Assistance

Visit the tech service page on our web site at [SigmaAldrich.com/techservice](#).
Any serious incident of this device should be reported to manufacturer and competent authority of country where user is established.

Standard Warranty

The applicable warranty for the products listed in this publication may be found at [SigmaAldrich.com/terms](#).

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