Instructions for Use

Sudan Black B Staining System

Procedure No. 380

Intended Use

Sigma-Aldrich Sudan Black B Staining System is intended for use as a general laboratory histochemical stain for the demonstration of neutrophil granules. The Sudan Black B Staining System is for professional "In Vitro Diagnostic Use" only. This manual, qualitative procedure identifies neutrophil granules and their precursors in blood or bone marrow films of human specimens. Histological visualization of neutrophil granules is a unique technique currently commonly used in medicine.

Several lipids including phospholipids, neutral fats and sterols are stained intensely by Sudan Black B. The reaction of neutrophil granules with the dye was described by Sheehan and Storey in 1947.¹ The leukocyte Sudan Black B staining pattern usually parallels that of myeloperoxidase.² Cells committed along lymphoid pathways display negative stain, whereas myeloid and monocytoid forms display characteristic positive reactions.

Previous methods utilized formaldehyde vapor fixation of blood films.² This technique may result in cell loss and staining artifacts. The Sigma-Aldrich procedure utilizes a buffered glutaraldehyde fixative and shortened incubation time that results in excellent staining without cellular loss or distortion.³

Reagents

Sudan Black B Staining Reagent (Cat. No. 3801-200ML)

Sudan Black B, C.I. 26150, 0.18% (w/v), in 69% ethanol and containing phosphate-buffered phenol.

Hematoxylin Solution, Gill No. 3 (Cat. No. GHS3-100ML)

Certified hematoxylin, C.I. 75290, 6 g/L, sodium iodate, 0.6 g/L, aluminum sulfate, 52.8 g/L and stabilizer. Danger. Harmful if swallowed. Causes serious eye damage. Wear protective gloves/eye protection/face protection.

Glutaraldehyde Solution (Cat. No. 3802-75ML)

Glutaraldehyde, 4% in borate buffer, pH 7.6

Special Materials Required but Not Provided

- Acetone, ACS Reagent

Storage and Stability

Store Sudan Black B Staining Reagent at room temperature (18–26°C). Reagent label bears expiration date.

Store Hematoxylin Solution, Gill No. 3 at room temperature (18–26°C) protected from light. Reagent label bears expiration date. Filter before use. It is recommended that material be returned to original container after use. Discard if solution turns brown (over oxidized by air) or purple (loss of acidity).

Store Glutaraldehyde Solution in refrigerator (2–8°C). Reagent label bears expiration date. Discard if turbidity develops.

Glutaraldehyde Fixative Solution is stable several months when stored in tightly capped glass bottle in the refrigerator (2–8°C).

Preparation

Prepare Glutaraldehyde Fixative Solution by adding 25 mL of ACS or reagent grade acetone to 75 mL Glutaraldehyde Solution.

Precautions

The IVDs included in these kits are intended for in vitro diagnostic use in a clinical laboratory environment. These IVDs are for professional use by qualified personnel only. Sigma-Aldrich IVDs may be operated by laboratory personnel who are trained to handle human specimens that can be infectious, use microscopes and other laboratory equipment and have color perception and visual acuity to distinguish colors and other objects under a microscope.

Normal precautions exercised in handling laboratory reagents should be followed. Dispose of waste observing all local, state, provincial or national regulations.

Procedure

Specimen Collection

No known test method can offer complete assurance that blood samples or tissue will not transmit infection. Therefore, all blood derivatives or tissue specimens should be considered potentially infectious.

Freshly prepared whole or anticoagulated blood or bone marrow films may be used. Fix as soon as possible.

Notes

It is recommended that blood films prepared from healthy donors be processed along with patient samples as a system-performance check.

Procedure

- Cool Glutaraldehyde Fixative Solution in refrigerator (2–8°C).
- Smears or films of blood or bone marrow preparations are fixed for 1 minute at 2–8°C with gentle agitation followed by thorough rinsing in deionized water.
- Stain in Sudan Black B Staining Reagent by immersion for 5 minutes with intermittent agitation.
- Rinse 3 or more times in 70% ethanol until no more dye washes out. This is followed by thorough rinsing in distilled water.
- Counterstain in Hematoxylin Solution, Gill No. 3, for 5 minutes followed by thorough rinsing in tap water.
- After air drying, slides may be mounted in DPX Mountant for histology or other suitable permanent mounting medium.

Performance Characteristics

Neutrophils and their precursors show blue-black intracellular granulation.² Monocytes stain less intensely and lymphocytes do not stain with Sudan Black B.²

Blood films prepared from normal donors were stained for Sudan Black B according to the described procedure and by the classic Sheehan-Storey method.¹ Neutrophils showed blue-black granulation with Sigma-Aldrich Procedure No. 380, and blue-black granulation with the Sheehan-Storey procedure. In both cases, monocytes stained less intensely and lymphocytes did not show myeloperoxidase activity.

If observed results vary from expected results, please contact Sigma-Aldrich Technical Service for assistance.

Analytical Performance Characteristics

The analytical performance results for the given tests conducted on all target structures, confirm 100% sensitivity, specificity, and repeatability.

Cat. No	Product Description	Target	Intra-assay Specificity	Intra-assay Sensitivity	Inter-assay Specificity	Inter-assay Sensitivity
3801	Sudan Black B Staining Reagent	Granulocytes	3 of 3	3 of 3	3 of 3	3 of 3
		Monocytes	3 of 3	3 of 3	3 of 3	3 of 3
GHS3	Hematoxylin Solution, Gill No. 3	Nuclei	3 of 3	3 of 3	3 of 3	3 of 3
3802	Glutaraldehyde Solution	Granulocytes	3 of 3	3 of 3	3 of 3	3 of 3
		Monocytes	3 of 3	3 of 3	3 of 3	3 of 3

Warnings and Hazards

Refer to Safety Data Sheet and product labeling for any updated risk, hazard or safety information.

380B:

H225 Highly flammable liquid and vapor.

H302 + H332 Harmful if swallowed or if inhaled.

H314 Causes severe skin burns and eye damage.

H317 May cause an allergic skin reaction.

H334 May cause allergy or asthma symptoms or breathing difficulties if inhaled.

H335 May cause respiratory irritation.

H341 Suspected of causing genetic defects.

H371 May cause damage to organs (Eyes, Central nervous system).

H373 May cause damage to organs (Kidney) through prolonged or repeated exposure if swallowed.

H411 Toxic to aquatic life with long lasting effects.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P273 Avoid release to the environment.

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.

P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.

P304 + P340 + P310 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/ doctor.

P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Symbol Definitions

Symbols as defined in EN ISO 15223-1:2021

	Manufacturer		Catalogue Number
	Consult Instructions for Use		Batch Code
	Authorized Representative in the European Community/ European Union		European Union Declaration of Conformity (defined in IVDR 2017/746)
	Use-by Date		In vitro diagnostic medical device
	Temperature Limit		Caution
	Date of Manufacture		Importer
	Indicates the Authorised Representative in Switzerland		

References

1. Sheehan HL, Storey GW: An improved method of staining leukocyte granules with Sudan Black B. J Pathol Bacteriol 59:336, 1947
2. Davey FR, Nelson DA: Sudan Black B Staining. IN Hematology, 2nd ed. WJ Williams, E Beutler, AJ Erslev, RW Rundles, Editors, McGraw-Hill, New York, 1977, pp 1629–1630
3. Harker JS, Laszlo J, Moore JO: The light microscopic demonstration of hydroperoxidase-positive phi bodies and rods in leukocytes in acute myeloid leukemia. Histochemistry 58:241, 1978

Contact Information

To place an order, please visit our web site at SigmaAldrich.com. For Technical Service, please visit the tech service page on our web site at SigmaAldrich.com/techservice.

Revision History

Rev. 4.0	2014
Rev. 5.0	2016
Rev. 6.0	2023
Transferred to new template with current branding. Specified for professional use in intended use and precautions. Revised intended use to align with IVDR guidelines. Updated Material Safety Data Sheet to Safety Data Sheet. Updated contact information. Removed instruction to follow CLSI for specimen collection. Removed EN 980 and changed to EN ISO 15223-1:2021 for symbols. Added adverse event contact information. Added Warnings and Hazards. Added CH-REP information.	

Sigma-Aldrich, Inc.,
3050 Spruce Street,
St. Louis, MO 63103 USA
an affiliate of Merck KGaA,
Darmstadt, Germany
+1(314) 771-5765

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Merck KGaA,
64271
Darmstadt,
Germany

MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

The Initial M and Sigma-Aldrich are trademarks of Merck KGaA, Darmstadt, Germany or its affiliates. All other trademarks are the property of their respective owners. Detailed information on trademarks is available via publicly accessible resources.

© 2023 Merck KGaA, Darmstadt, Germany and/or its affiliates. All rights reserved.

The life science business of Merck operates as MilliporeSigma in the U.S. and Canada.

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Referenzen

- 1. Sheehan HL, Storey GW: An improved method of staining leukocyte granules with Sudan Black B. J Pathol Bacteriol 59:336, 1947
- 2. Davey FR, Nelson DA: Sudan Black B Staining. IN Hematology, 2nd ed. WJ Williams, E Beutler, AJ Erslev, RW Rundles, Editors, McGraw-Hill, New York, 1977, pp 1629–1630
- 3. Hanker JS, Laszlo J, Moore JO: The light microscopic demonstration of hydroperoxidase-positive phi bodies and rods in leukocytes in acute myeloid leukemia. Histochemistry 58:241, 1978

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Bibliografia

1. Sheehan HL, Storey GW: An improved method of staining leukocyte granules with Sudan Black B. J Pathol Bacteriol 59:336, 1947.
2. Davey FR, Nelson DA: Sudan Black B Staining. IN Hematology, 2nd ed. WJ Williams, E Beutler, AJ Erslev, RW Rundles, Editors, McGraw-Hill, New York, 1977, pp 1629–1630.
3. Harker JS, Laszlo J, Moore JO: The light microscopic demonstration of hydroperoxidase-positive phi bodies and rods in leukocytes in acute myeloid leukemia. Histochemistry 58:241, 1978.

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Referencias

1. Sheehan HL, Storey GW: An improved method of staining leukocyte granules with Sudan Black B. J Pathol Bacteriol 59:336, 1947
2. Davey FR, Nelson DA: Sudan Black B Staining. IN Hematology, 2nd ed. WJ Williams, E Beutler, AJ Erslev, RW Rundles, Editors, McGraw-Hill, New York, 1977, pp 1629–1630
3. Hanker JS, Laszlo J, Moore JO: The light microscopic demonstration of hydroperoxidase-positive phi bodies and rods in leukocytes in acute myeloid leukemia. Histochemistry 58:241, 1978

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Referenser

1. Sheehan HL, Storey GW: An improved method of staining leukocyte granules with Sudan Black B. J Pathol Bacteriol 59:336, 1947
2. Davey FR, Nelson DA: Sudan Black B-färgning. IN Hematology, 2:a utg. WJ Williams, E Beutler, AJ Erslev, RW Rundles, Redaktörer, McGraw-Hill, New York, 1977, sid. 1629–1630
3. Harker JS, Laszlo J, Moore JO: The light microscopic demonstration of hydroperoxidase-positive phi bodies and rods in leukocytes in acute myeloid leukemia. Histochemistry 58:241, 1978

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MDSS GmbH